



X-RAY alignment

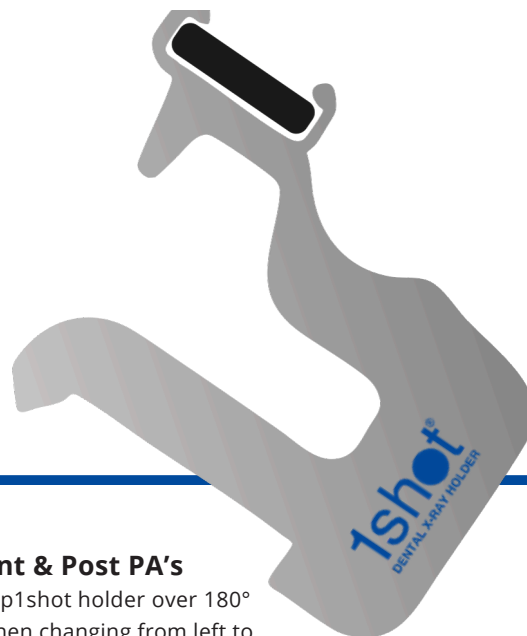

Bite Wing X-Ray


Align in three planes

1. At right angles to near edge
2. Through centre of target
3. Through flat plane of holder

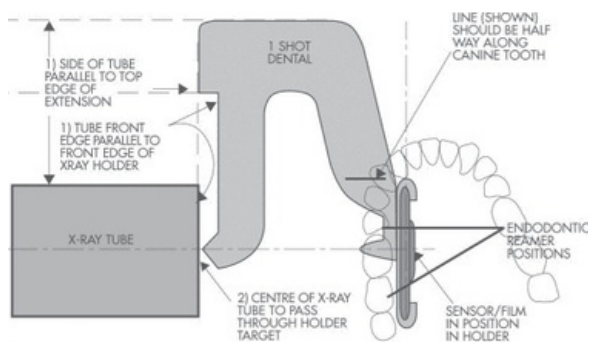


Ant & Post PA's

Flip 1shot holder over 180° when changing from left to right side of mouth.

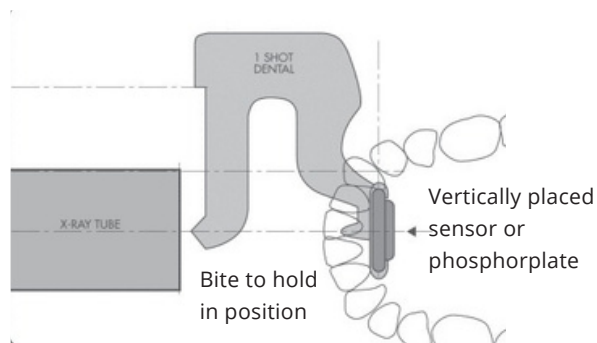


PA alignment

PA's- lower or raise both X-Ray tube and phosphorplate or sensor

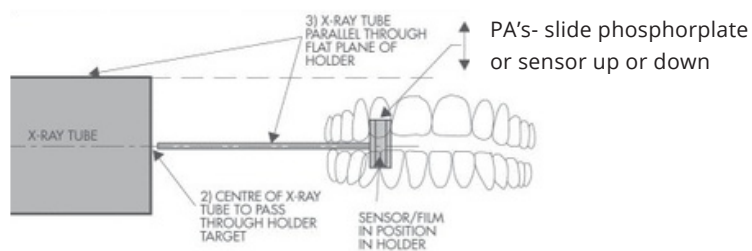


Endodontic - PA's Ant & Post

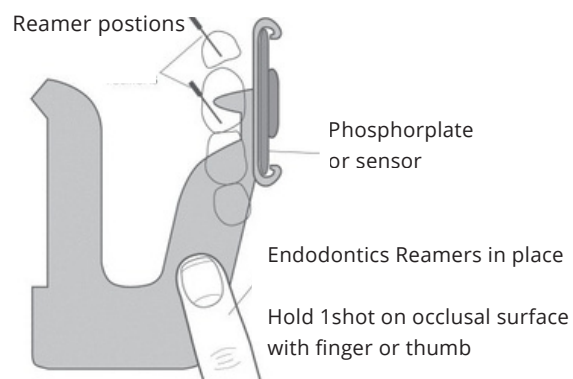


Assembly 1shot Phosphorplate or Sensor

Horizontal

Vertical

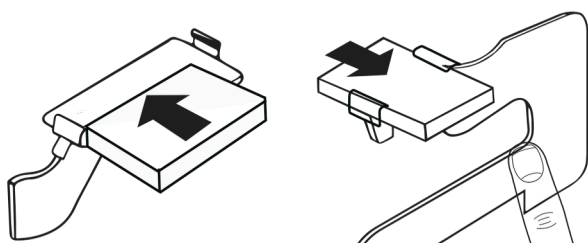

Adjusting 1shot claw for thicker sensor

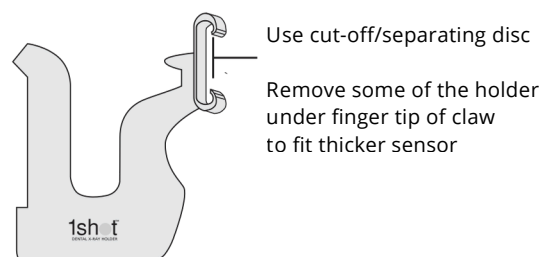


SUB-Series assembly

Universal & Emergency holders (U&E holders)



AUTOCLAVE

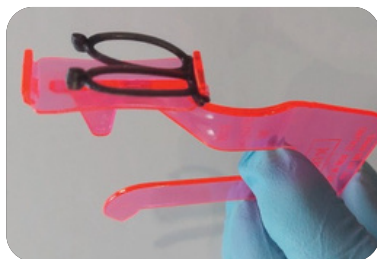


Autoclave with SUB (U&E) bands in place as in picture.

Identify one of four 1shot SUB (U&E) holders required to perform desired shot.

PhosphorPlate / Film & Sensor PLACEMENT

Horizontal placement

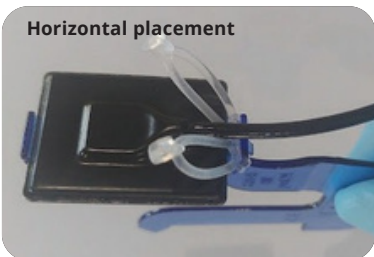


Assembly guide



Place the sensor between the SUB bands and onto the platform.

Hook the 2 SUB bands over the Phosphorplate or Sensor and secure to the other side of the platform.

To remove simply unhook the bands then remove the Phosphorplate or Sensor.

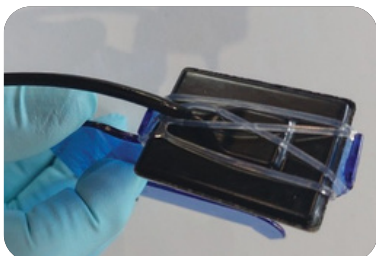

Vertical placement

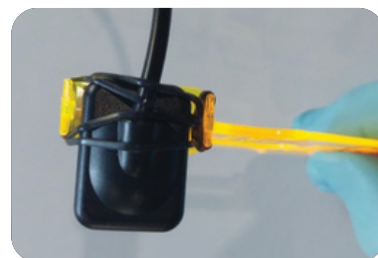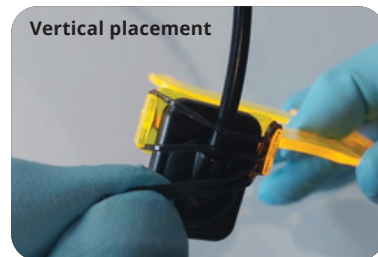

Please scan QR code with your smartphone or iPad to watch our videos for Phosphorplates and SUB Sensor assembly and removal.



Manufactured & distributed by
Top Operator Products - 1shotdental
Unit 2, 88 Maungakawa Rd, RD4,
Cambridge 3496
New Zealand

www.1shotdental.com | 1shot® is a USA registered trademark.